

& Serial # of unit on back of unit: **NEDS**

7. Press the number **"0"** 3 times to exit screens.

**Step
2**

Step # 2, Reporting ID number and requesting code for additional usage.

Go to the NeuroMed International website, and select the phase usage request and fillout the request form. Please provide us with the serial number of the unit (NEDSXX-XXXXX) and report the ID number and the number of additional phases/usages/treatments you wish to upload. NeuroMed will use this information to generate a **Special Code** number for you to enter into the NeuroMed Matrix unit. This procedure will take a little bit of time so, we request for you to do this at the end of your day of work and by the next morning you will receive the information in your e-mail in-box.

www.neuromedinternational.com


**Step
3**

Step # 3, Input of the additional usage and code number.

To enter the additional uses and the **Code** number supplied by the NeuroMed staff, use the following steps:

- Start with the **POWER** turned **OFF**.
- Set the **DOSAGE** to the **RESET** position.
- Turn the **POWER ON**.

Wait for the following screen to appear:

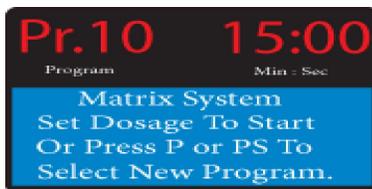

1. Press the **'ENTER'** key to get to the **Whose Option?** Screen:



2. Press the number **"1"** key to get to the **User** Option Screen:



3. You are now prompted for a 4-digit password. Enter the numbers: **“1” “2” “1” “4”**

Navigate to the **User Option** screen as before:



4. Select option **“3”: Add usages** to display the **Matrix System** Screen



Enter the number of treatments at the prompt **Add Usage Times:** 
This must be the same number as reported to the NeuroMed staff earlier.
Now enter the **Code number** supplied by the NeuroMed staff at the **Code=**

 prompt.

The screen changes automatically and displays a new **ID** number and the updated **Usage Left:** number

Press **“ENTER”**



4. Enter **“0”** three times.

5. Turn off the **Power** and back on again prior to the next treatment.